

IMPORTANT!

PLEASE READ THESE
INSTRUCTIONS BEFORE FIRST USE.

ONLY USE "312" SIZE HEARING AID
BATTERIES.

INSERT BATTERIES CORRECTLY -
DO NOT FORCE THEM!

BEFORE RETURNING: CALL US
TO RECEIVE AN RMA (RETURN
MATERIAL AUTHORIZATION).

**DO YOU NEED HELP?
CALL TOLL FREE**

1-877-670-0745

QUICK START GUIDE

Get the most out of your Perfect Choice! This quick start guide is a brief overview. Please read the enclosed instruction book for further details on operation and care.

Make sure your ears are free of wax. Earwax is the leading cause of device failure.

Remove the tube from the unit before cleaning.

Regularly clean the tube using the enclosed cleaning string.



Step 1

Battery Placement & Power On



Pull battery door open using your fingernail.



Remove tab from battery to activate.



Place battery into door with + sign facing out. The magnet tool may help to pickup or remove your battery.



Close battery door to power device on. Wait 5 seconds for device to start up.

Step 2

Volume Adjustment



Place Perfect Choice behind your ear and the tip of the tubing in your ear. Volume control starts in the middle of a ten step range.



To Increase Volume

Press the upper part of the volume control. Each press will increase the volume slightly. A male voice will confirm your selection.



To Decrease Volume

Press the lower part of the volume control. Each press will decrease the volume - confirmed by a male voice.

Perfect Choice remembers the last volume and program settings even after you remove the battery.

Step 3

Listening Programs



To switch between programs, press and release the small button at the top of the unit. A female voice will confirm your selection.

Programs

Program 1: Listening / Conversation

Program 2: Voice Boost / Warmer sound;
Less tinny.

Program 3: Crowded Rooms & Noisy Situations: Directional Microphones only amplify sound coming from in front of you thereby reducing background noise.

Powering OFF




Use your fingernail to open the battery door. Leave the door open. the battery can remain in the device. Store in a safe and dry place.

Step 4 Sizing


Tips & Tubing Change



To remove the tube, gently rotate the sound tubing holding the instrument firmly. Pull the sound tubing away as you twist. Press the tubing onto the device to replace.

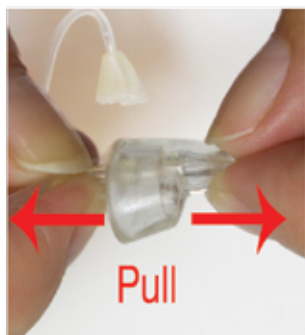

Remove the Dome Tip or ZipTip by pulling it off of the tubing. The round dome tip is for smaller ears and to help reduce any “plugged-up” feeling.

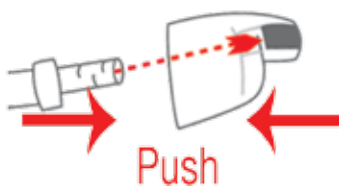

Vent hole
towards bottom
of ear canal

Replace the Dome Tip or ZipTip by lining up the end of the tubing with the top hole and gently press until you feel the tubing “click.” The bottom hole is used for venting. Do not plug this hole.